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## For In Vitro Diagnostic Use

### Intended Use

IMPROVIRAL Viral Preservative Medium (VPM) for viral specimen is intended for the collection and transportation of clinical specimens containing viruses from the collection site to the testing laboratory. It maintains the viability of the virus and integrity of the sample.

### Application

IMPROVIRAL VPM is suitable for hospitals, disease control centers, independent laboratories, medical examination centers, scientific research institutes and other institutions conducting viral research, diagnosis, and treatment. For biosafety reasons, testing needs to be performed in specialized laboratories for biosafety management by professional users.

### Composition

IMPROVIRAL VPM is mainly composed of RPMI 1640 media formulation, buffer solution, Gentamicin, Penicillin and Streptomycin.

### Specimen Collection and Handling

1. After the swab specimen is collected from patient, insert it vertically into the IMPROVIRAL VPM tube.
2. Elute the swab specimen in the medium or break the swab in the tube.
3. Recap the tube of IMPROVIRAL VPM clockwise to avoid leakage.
4. If the test cannot be performed immediately, store the tube with specimen upright under 0-8°C and transport it to the testing laboratory within 48 hours.

### Cautions and Warnings

1. Please read the instruction manual carefully before use.
2. The preservation solution should be homogeneous and clear, without visible impurities or precipitation.
3. The swab specimens should be inserted vertically into the IMPROVIRAL VPM tube without touching other parts to avoid contamination.
4. Dispose of used swabs, swab sticks, IMPROVIRAL VPM and IMPROVIRAL VPM tubes in accordance with local medical waste regulations.
5. After specimen collection, recap the tube of IMPROVIRAL VPM clockwise to avoid unexpected leakage.

### Storage

The recommended storage temperature is 0-8°C and no more than 30 days at room temperature, avoiding direct exposure to sunlight.

### Shelf life

The shelf life is 12 months from the production date.

Guangzhou Improve Medical Instruments Co., Ltd.  
No.102, Kaiyuan Avenue, Science City, Guangzhou Economic & Technological Development District, Guangzhou, 510530, CHINA  
www.improve-med.com

### Authorized EC Representative

Shanghai International Holding Corp. GmbH (Europe)  
Eiffestrasse 80, 20537 Hamburg, Germany

Revision and date of the instructions for use Revision: A/2 Date of issue: 2020.05.15

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Shanghai International Holding Corp. GmbH (Europa)